

VEINPLUS®

the new treatment

Venous Insufficiency

Aching, painful, “heavy” legs

Venous disorders during pregnancy

Swollen legs or ankles

Post-Thrombotic Syndrome (PTS)

Situations at risk for your veins

...



Ask your health professional or go to


www.veinoplus.com

VEINOPLUS®


main clinical results:

REDUCTION OF PAIN IN LEGS

Pain in legs in a population of patients with venous insufficiency ¹

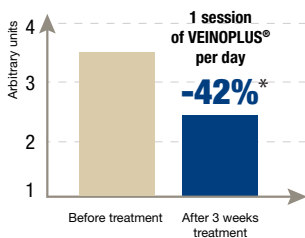


Pain in legs in a population of pregnant women with venous insufficiency ²

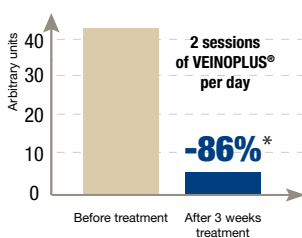

REDUCTION OF EDEMA

Reduction of Ankle Edema in 30 patients (CEAP classification C3) after a 30 day treatment with VEINOPLUS®³

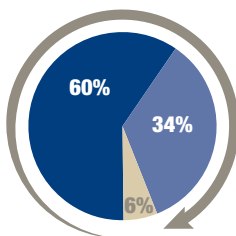


94% of cases

- Total reduction
- Partial reduction
- No reduction

IMPROVEMENT OF QUALITY OF LIFE

Reduction of negative impact of venous insufficiency ² in a population of 38 pregnant women

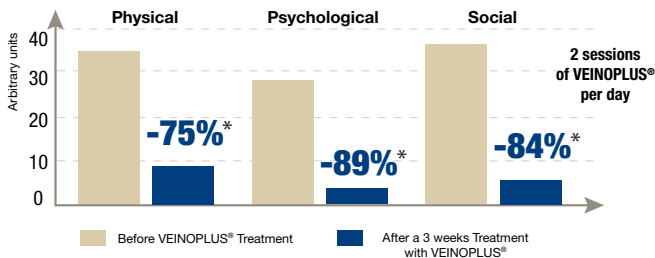

1. Zuccarelli F, Launay J, LeMagrex J, Mollard R, Fargier P, Pujo M. Activation of Calf Muscle Pump by VEINOPLUS® Stimulation (Fr). *Angéiologie* 2005;57:48-54. • 2. LeTohic A, Bastian H, Pujo M, Beslot P, Mollard R, Madlenat P, Effect of electro-stimulation (Veinoplus) on lower limbs venous insufficiency-related symptoms during pregnancy. *Gynecol Obstet&Fertil* 2009;37:18-24 (data from group N°2) • 3. BOGACHEV V.Y. et al. : Neuromuscular stimulation with VEINOPLUS® - a new method for the treatment of Chronic Venous Insufficiency in the lower limbs. In *PHLEBOLOGY* (Russ. Assoc. of Phlebology & IUP) 2010; Vol; 1, Issue 4 Pages 22-25.

* significant p < 0,001

**CEAP : International Classification for Chronic Venous Disorders (C3 = Edema)

VEINOPPLUS®

is recommended for:

■ Symptoms of Venous Insufficiency

- Aching, painful, “heavy” legs
- Pregnancy related venous disorders
- Swollen legs or ankles
- Edema of lower legs*
- Night cramps*
- Restless legs*
- Post-thrombotic syndrome
- Venous ulcers**

■ Situations at risk

- Prior episode of Deep Vein Thrombosis (DVT)
- Professions at risk (waiters, flight crew...)
- Prolonged sitting or standing
- Leg Immobilisation (plaster...)
- Long haul flights (> 4h)
- Lack of exercise
- Excessive weight, obesity

**88% satisfaction
in our latest
users survey*****



* in case of venous origin ** with doctor's consent

*** survey on 100 patients

VEINOPLUS®

What is VEINOPLUS® ?

VEINOPLUS® is a battery powered electrical muscle stimulator (CE medical: class IIa). VEINOPLUS® was invented by Dr. Jozef Cywinski, one of the world experts in electrostimulation.

VEINOPLUS®... an efficient technology

VEINOPLUS® stimulates the “calf pump” which is responsible for most of the physiological blood return to the heart. Thanks to this action VEINOPLUS® has a unique capability of therapeutic effects on the symptoms of Venous Disease (from a simple pain in legs to more serious conditions).

These effects* have been demonstrated and documented in several clinical studies made since 2004 in France, UK, USA and Russia.

VEINOPLUS®... a safe technology



VEINOPLUS® meets the most stringent safety standards established in the USA (norm AAMI/ANSI NS-4 1986/2002). VEINOPLUS® can be used even during pregnancy as shown in a clinical study made in France¹.

Using VEINOPLUS®... nothing easier!

- put one electrode on each calf if symptoms are bilateral (or put 2 electrodes on the same calf if symptoms are on one side only)
- set up intensity until you feel a strong yet comfortable calf contraction
- ... the device stops automatically after 20 minutes.

When using VEINOPLUS®?

Depending upon your symptoms*:

- minimum once per a day when starting the treatment,
- then when you feel you need it, no risk of overuse!
- ... the more you use VEINOPLUS® the more effective it becomes.

Is there any contra-indication?

People with implanted cardiac pacemakers must not use the device. For other warnings, please refer to the user's manual. In case of doubt ask your health professional.

Any question?

Go to our website www.veinoplus.com or the one of your local distributor.

* effects are not guaranteed, can vary from person to person and depend upon symptoms.
1 - Le Tohic A. et al., Effect of electro-stimulation (VEINOPLUS®) on lower limbs... (cited reference)

Manufactured by:

Ad Rem Technology

162, rue du Faubourg Saint-Honoré 75008 Paris - FRANCE

Tel: +33 (0)1 42 60 00 22 - Fax: +33 (0)1 42 60 00 63 - www.adremtechnology.com

ISO 13485:2003 & ISO: 9001:2008

Distributed by: